



VisAR Quick Start Workflows

Launch a study from the QR code, or the Built-In images tab to begin.

REGISTRATION WORKFLOW



Say **"REGISTRATION WORKFLOW"**

This will guide you through these steps.

- Find any landmarks
- Find any skin tags
- Register any patient trackers

If none of these are found, it will default to manual registration.


NAVIGATION WORKFLOW



Say **"NAVIGATION WORKFLOW"**

This will guide you through these steps.

- Set Instrument Length
- Start Patient Tracking
- Show Navigation Views
- Start Instrument Tracking

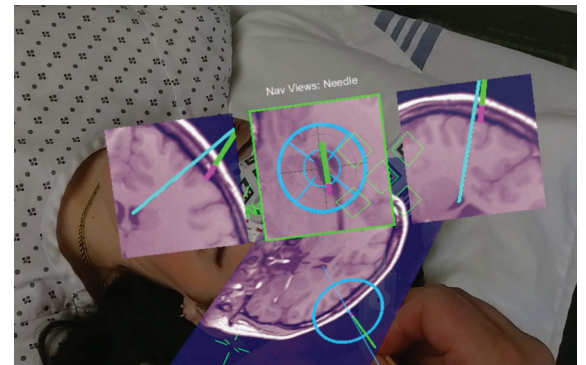

Align the green needle of the instrument with the trajectory of the blue virtual needle.

CALIBRATION WORKFLOW



Say **"CALIBRATION WORKFLOW"**

This will guide you through these steps.

- Room calibration
- Eye Calibration


CREATE NEEDLE WORKFLOW



Say "CREATE NEEDLE WORKFLOW"

This will guide you through these steps.

- Show the closest slice in an orthogonal plane.
- Drag thru slices to find where you want needle.
- Say "Stop dragging" at desired slice.
- Say "Create Needle."
- Drag needle tip.
- Drag needle end.

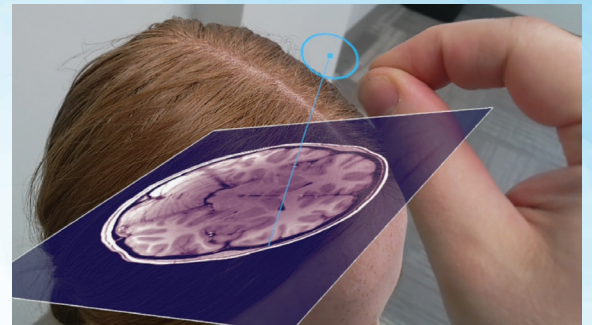

Create a needle by pinching and dragging virtual needle tip/end

PEDICLE WORKFLOW



Say "PEDICLE WORKFLOW"

This will guide you through these steps.

- Find the pedicle with surface view.
- Create a pedicle screw using combined 2D Nav Views and 3D View.
- Adjust the pedicle screw position, length, and diameter.

CUSTOM WORKFLOW



Say "CUSTOM WORKFLOW"

This allows a virtual tour of various features.